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EVIDENT SUCCESS

DENTSPLY

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The TissueCare Concept:
**A new definition of
tissue stability**

For better dentistry

DENTSPLY

A new definition of tissue stability.

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X-ray image of the Ankylos implant-abutment connection before start of the test (x-ray image: Holger Zipprich/Dr. Paul Weigl, Frankfurt am Main)

X-ray image of the Ankylos implant-abutment connection after 1,000,000 load cycles (x-ray image: Holger Zipprich/Dr. Paul Weigl, Frankfurt am Main)

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Literature:

1. Abboud M, et al. Immediate loading of single-tooth implants in the posterior region. *Int J Oral Maxillofac Implants*. 2005 Jan-Feb;20(1):61-8.
2. Zipprich H et al. Failure mode of implant-abutment connections after horizontal cyclic loading, *Int Poster J Dent Oral Med* 2004, Vol 6 No 03, Poster 238.
3. Dibart S, Warbington ML, Su MF, Skobe Z. Evaluating the Bacterial Seal of an Implant-Abutment Connection. Poster presentation on the American Academy of Periodontology Meeting, Orlando, Florida in November, 2004.
4. Sethi A, Kaus T. An implant that does not smell - The Ankylos Implant. Featured in *Irish Dentist*-July 2002; *Probe*-July 2002; *Dentistry*-August 2003.

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A bacteria-proof friction-locked connection is formed only if the taper is flush over its entire surface.

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Platform-Switching combined with a stable, bacteria-proof connection design is an important factor for tissue stability (x-ray image: Dr. Nigel Saynor, Stockport, UK)

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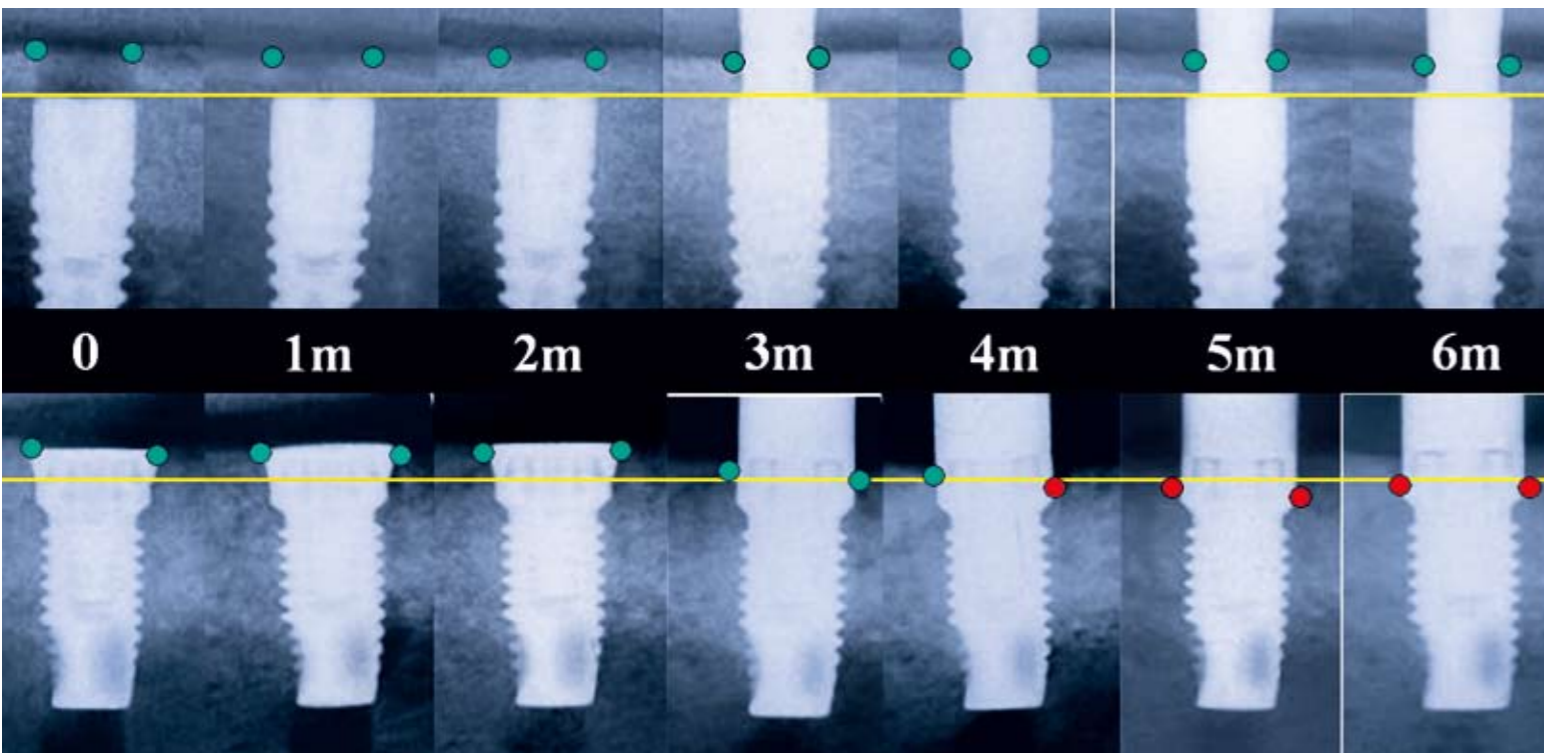
Deposition of bone tissue on the implant face as a result of subcrestal placement. Status three months after uncover (histology: Dr. Dietmar Weng, Starnberg)

In the course of a radiographic and histological animal study the implant shoulders were placed 1.5 mm below the bone margin in the front tooth region in accordance with an esthetic indication to simulate the submerged position of the implant-abutment connection site in the area of the bone support of the papillae.

*Standardized X-ray images were taken at monthly intervals. In addition to other results the peri-implant bone margin around the tapered connection remained significantly higher than that of a blunt abutment connection with an external hexagon over the six months of the trial.
(X-ray images: Dr. Dietmar Weng, Starnberg)*

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Literature:

1. Sammons RL, et al. Comparison of osteoblast spreading on microstructured dental implant surfaces and cell behaviour in an explant model of osseointegration. A scanning electron microscopic study. Clin Oral Implants Res. 2005 Dec;16(6):657-66.
2. Di Iorio, D., T. Traini, et al. (2005). "Quantitative evaluation of the fibrin clot extension on different implant surfaces: An in vitro study." J Biomed Mater Res B Appl Biomater 74(1): 636-42.
3. Degidi et al. "Immediately loaded titanium implant with a tissue stabilizing/maintaining design ("beyond platform switch") retrieved from man after 4 weeks: a histological and histomorphometrical evaluation: A case report" COIR 2007, accepted for publication.

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*1 | Stable peri-implant hard and soft tissue after uncover. 2 | 24 months after prosthetic restoration.
3 | 48 months after prosthetic restoration. 4 | Clinical situation (photos: Dr. Nigel Saynor, Stockport, UK).*